



## Universal IVF Medium

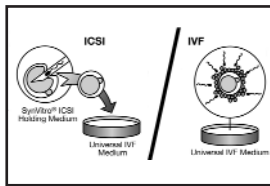


1. Carry out insemination (Day 0) in pre-equilibrated Universal IVF Medium. Where ICSI is required, the oocytes are transferred to Universal IVF Medium immediately after injection.

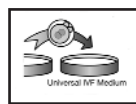

2. At 16-20 hours (Day 1), check for formation of pronuclei and transfer the pre-zygotes to a pre-equilibrated culture dish containing 50  $\mu$ l microdrops or 0.5 ml wells/dishes of Universal IVF Medium for further culture.  
*The embryos should be cultured singly or in multiples to a maximum of 4 per well.*

The embryos can be kept in the same medium from Day 1 until Day 3, and therefore no extra media changes are needed for Day 3 transfer compared to Day 2 transfer.

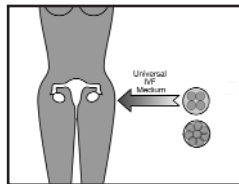

3. The embryos are prepared and transferred to the uterus in Universal IVF Medium at Day 2 when they have reached the 4-5-cell stage, or at Day 3, when they have reached the 6-8-cell stage.